

## PolyConversions, Inc.

3202 Apollo Drive, Champaign, IL 61822-9668 info@polycousa.com • www.polycousa.com Tel: 888 893 3330 • Fax: 217 893 3003

## **PolyGowns: Instructions for Use**Intended Use: This product is intended to be used as a personal protective apparel device to protect the wearer

Intended Use: This product is intended to be used as a personal protective apparel device to protect the wearer from the transfer and splash of materials and contaminates in the wearer's environment during low or minimal risk situations. This product is effective and appropriate when properly used according to the following instructions.

## **Donning**



1. Unfold gown.



2.Ensure gown is at full length.



3. Place arms through sleeves ensuring thumbs are inserted through thumb loops.



4. Place head through neck opening.



5. Untack gown ties ensuring that they are at full length.





6. Gown is ready to be tied in back or ties can be pulled around in front of gown and tied.



7. Put on gloves over sleeves for final step. (Note: Gloves are not supplied with gowns.)

## **Doffing**



1. Separate gown at shoulders by tearing along back perforation.





2. Release gown ties around waist by untying or pulling gown forward to force separation of tie from around waist.



3. Remove arms from sleeves and hands from gloves.



4. Gown and gloves are ready to be disposed of in an environmentally safe and appropriate manner.