

PolyConversions, Inc.

3202 Apollo Drive, Champaign, IL 61822-9668 info@polycousa.com • www.polycousa.com Tel: 888 893 3330 • Fax: 217 893 3003

VR® Gown: Instructions for Use Intended Use: This product is intended to be used as a personal protective apparel device to protect the wearer from the transfer and splash of materials and contaminates in the wearer's environment during low or minimal risk situations. This product is effective and appropriate when properly used according to the following instructions.

Donning



1. Unfold gown.



2.Ensure gown is at full length.





3. Place arms through sleeves. (Note: Ensure thumbs are inserted through thumb loops if using a thumb loop gown.)



4. Place head through neck opening.







5. Gown is ready to be tied in back. Put on gloves over sleeves for final step. (Note: Gloves are not supplied with gowns.)

Doffing





3. Remove arms from sleeves and hands from gloves.v



2. Release gown ties around waist by untying from back.



4. Gown and gloves are ready to be disposed of in an environmentally safe and appropriate manner.