

VR® Sleeve Gloves: Instructions for Use

Intended Use: This product is intended to be used as a personal protective device during cleaning processes. This product may only be used according to the instructions as described.

Donning



1. Grab the elastic end of the sleeve, insert hand and begin gently pulling sleeve until the hand is inside the glove cuff.



2. Grasp the cuff portion of the glove and pull onto to hand. Insert fingers completely into each void.



3. After all fingers are fitted into the glove, adjust sleeve portion to desired position on the arm.

Doffing



1. Pull on glove fingers and thumb until the palm of the hand is at the glove wrist.




2. Pull the fingers of the opposite glove so that both hands are in the palm or cuff area of the gloves.



3. Grasp and pull the fingers of the other glove until the wrist reaches the end of the sleeve. Repeat with the opposite hand.



4. With both hands at the elastic cuffs the sleeves are ready for removal.

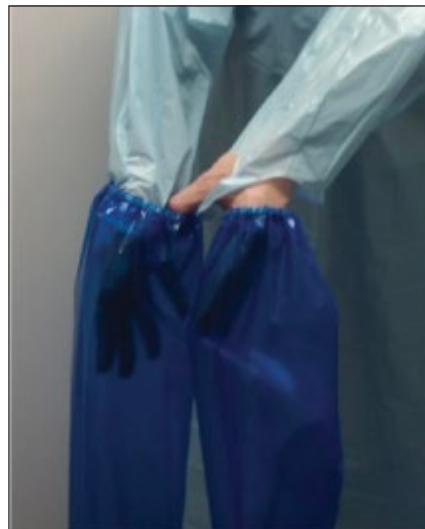

5. Grasp the inside of the opposite sleeve with thumb and forefinger to remove both sleeve gloves.



6. Sleeve gloves are ready to be disposed of properly.